Helpful Hairbrush

Assistive hairbrush designed specifically for the limitations of Fibrodysplasia Ossificans Progressiva (FOP)

Independently brush hair with reduced mobility


Design Criteria

 Use interdisciplinary teamwork to design a hairbrush specifically for the needs of those with FOP

WILL Project


"We Improve Life & Living with FOP"



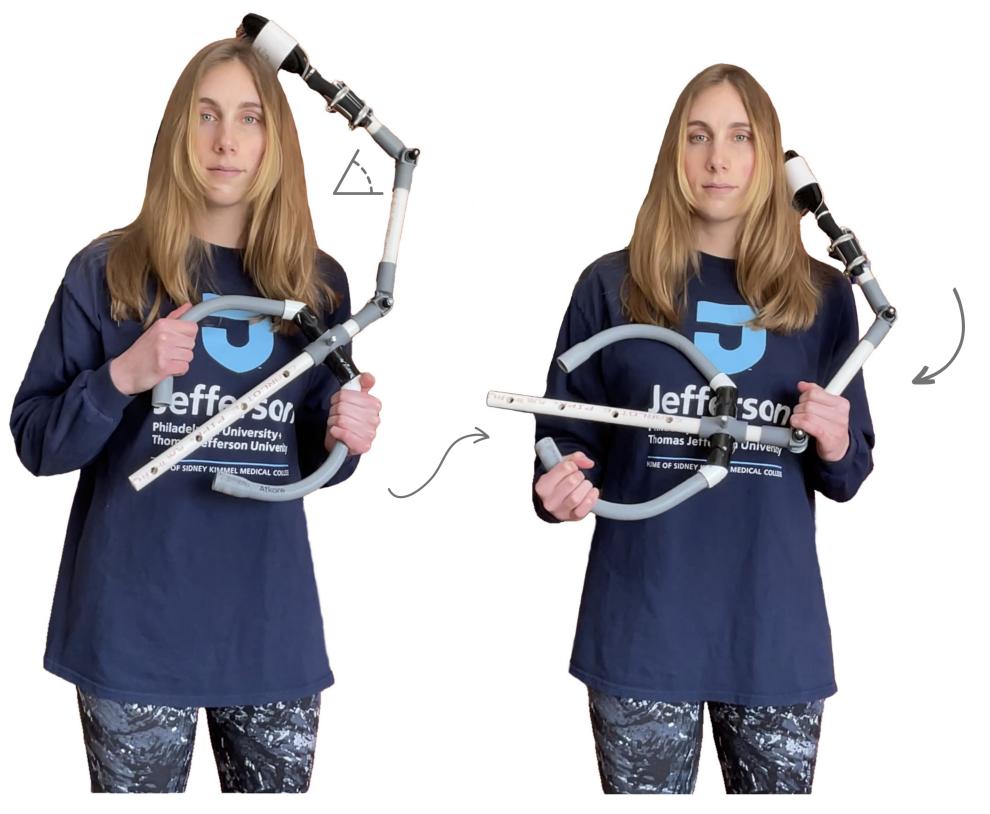

What is FOP?

Fibrodysplasia Ossificans Progressiva

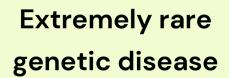


Muscles, tendons, and ligaments turn to bone

Process of progressive, uncontrolled bone growth brought on by "flare-ups"

Typical Skeleton
Separate bones




FOP SkeletonConnective tissues calcify & fuse together

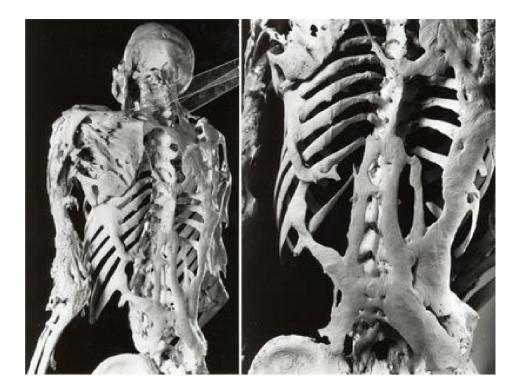

Cycle of flare-ups lead to permanent fixation of most major joints in the body

Progression Example:

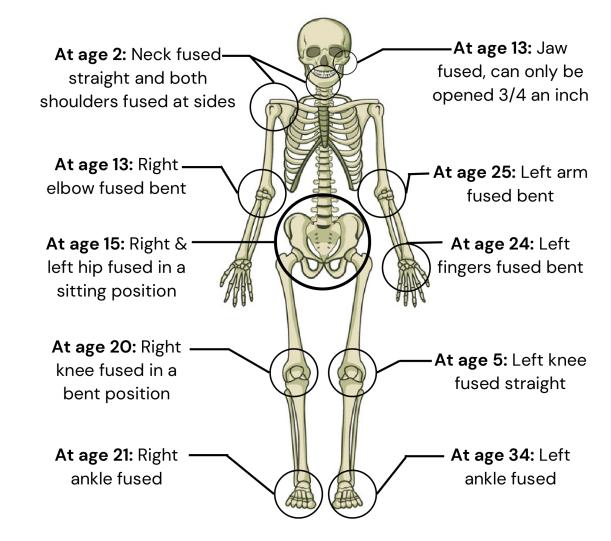



Daily Activities with FOP

Limited Mobility Impacts:

Individuals with FOP become fused and locked in positions

Limits independence

Makes daily activities difficult to execute.

- Reaching
- Toileting
- Oral Hygiene
- Eating
- Dressing
- Grooming

The progression and pattern of FOP is highly variable for each person. Current assistive devices do a poor job meeting their continual adaptations.

Self-Grooming Market








Current devices
lack the
necessary range
of movement to
brush your hair
with FOP

- Current assistive hairbrushes replicate an extension of the forearm
- Elongated handles and a slight curve to help reach the back of the head
 - Do not bend, turn, or adjust

Early in FOP disease progression, the **neck fuses** (fixed looking straight) and the **shoulders fuse** (arms fixed to sides)

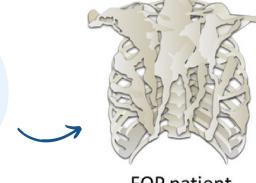



Healthy

Brushing Different Lengths of Hair


Typical movement has wrist & elbow in opposition: as wrist brushes down, elbow raises up

How might we design a hairbrush that...

- Effectively brushes long and short hair
- Can be used with shoulders fused at sides
- Doesn't put stain on the wrist


Top of Hair

- Forearm pivots at elbow
- Shoulder in; elbow in

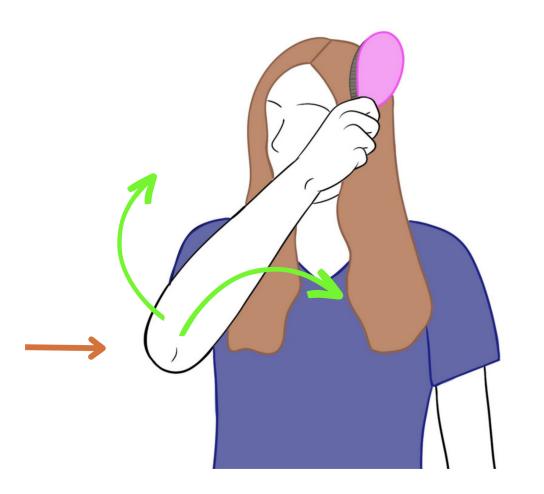


Midsection of Hair

- Forearm pivots at elbow
- Shoulder extended; elbow out

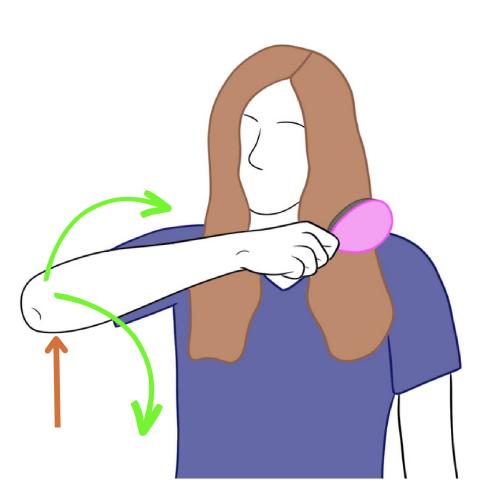


Ends of Long Hair

- Elbow & wrist pivot as user brushes down
- Shoulder & elbow move down with the brush

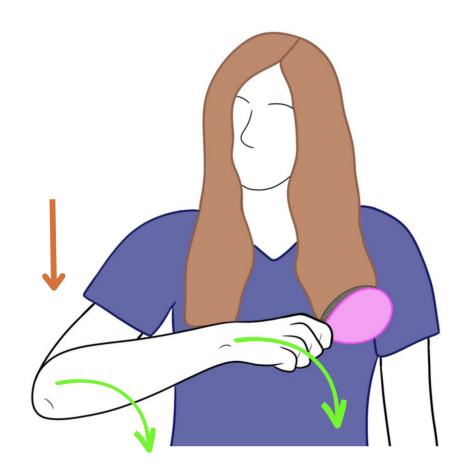


Exploring Lengths & Angles

 Different ways to brush long hair with FOP constraints, focusing on reaching the back of the head (most difficult spot)

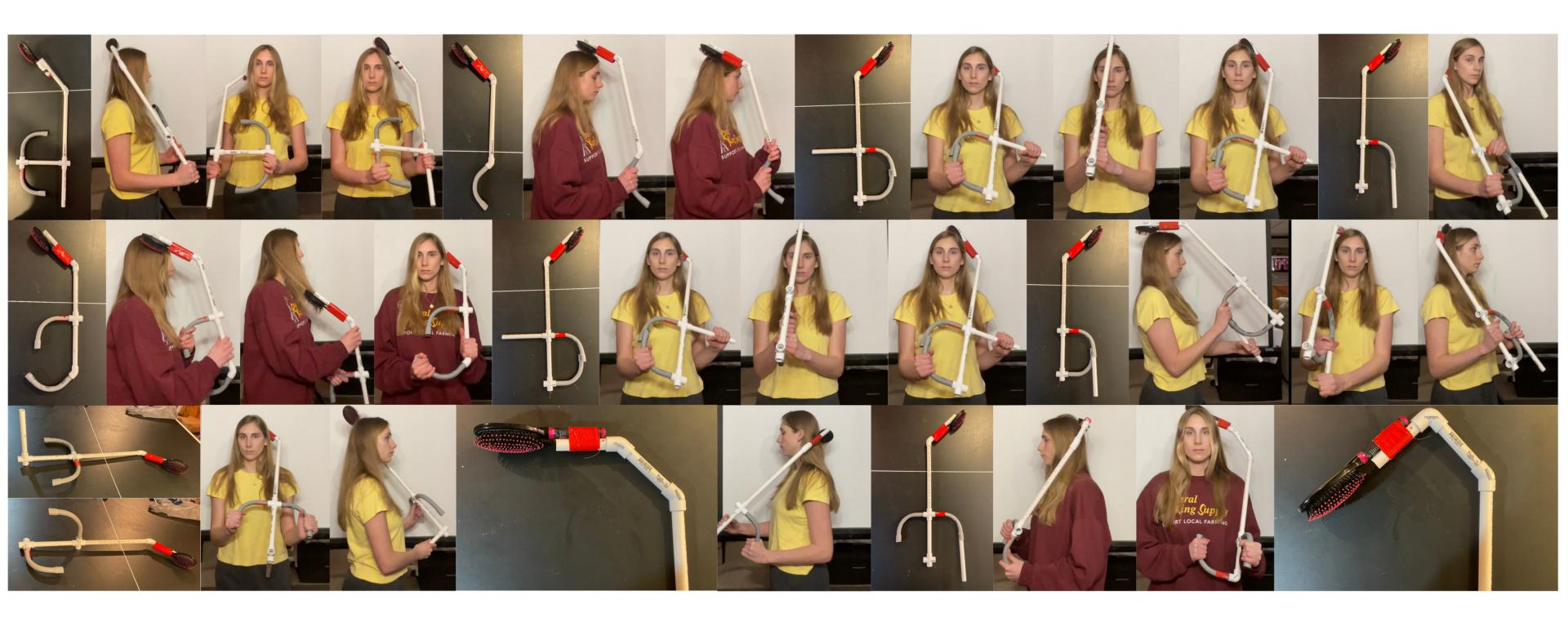




- Discovered that a "steering wheel" approach offered the greatest number of possible grip positions
- This design gives patients more options to test, providing more valuable feedback

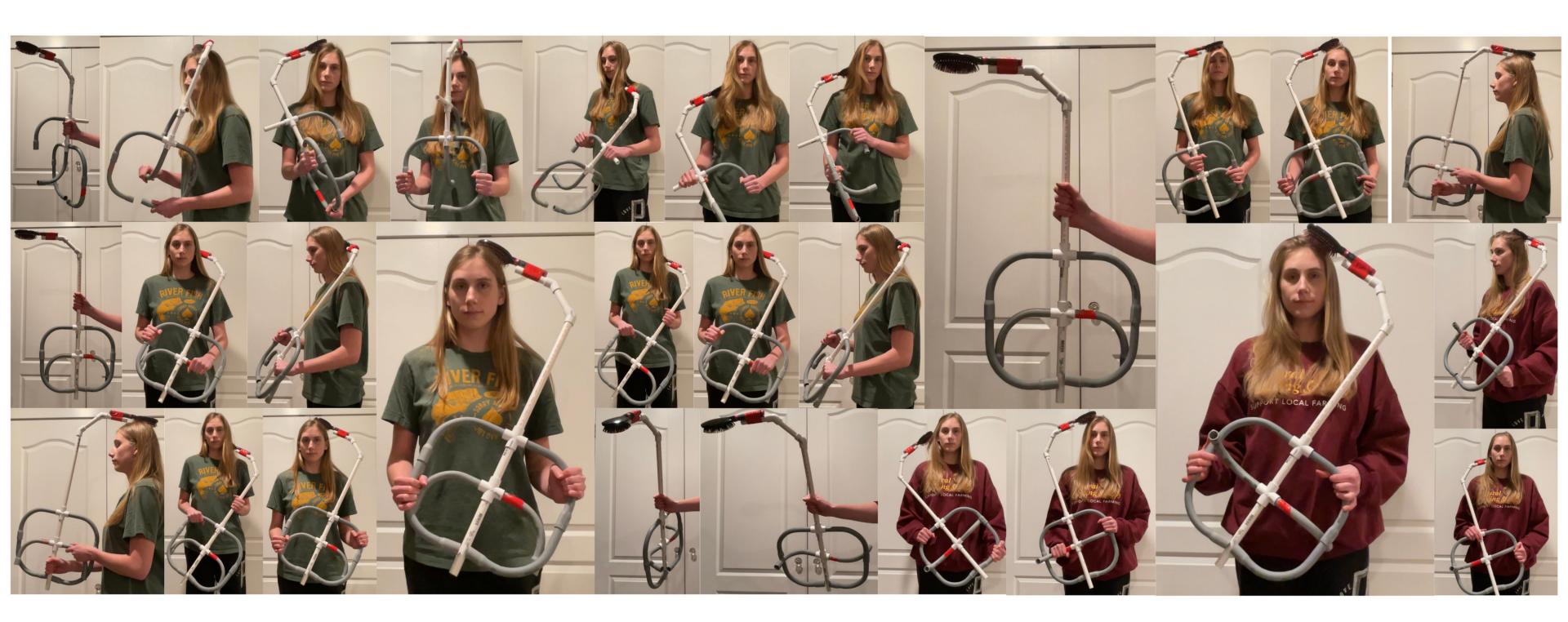

Sending Prototypes for User Testing

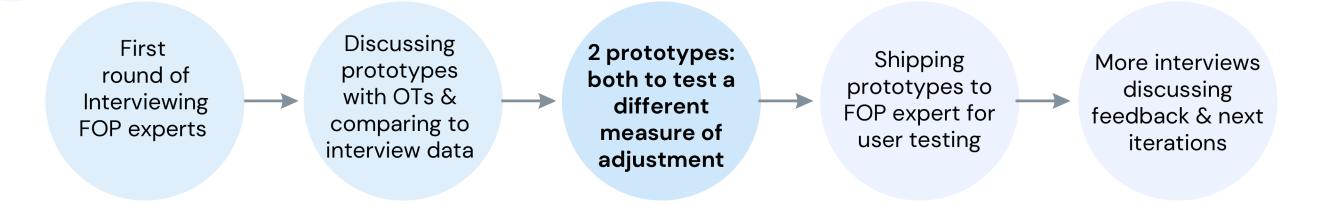

Proof of concept functional prototype

Exploration of heights & angles that accommodates the variable range of angles that the elbows can be fused at

Features

3D printed cuff

reinforced with

rubber

Clamps

hairbrush



3D printed hinge knuckles for adjustable angles

Adapts to PVC & secured with cam clamp




Prototype #1 Prototype #2

Jennifer Hegelein

Prototype #1


• "T" shaped handle design to test vertical grip options


Shaft height adjusts telescopically

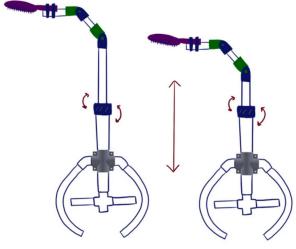

WWW.XM

Locks & unlocks with a twist, infinitely adjustable

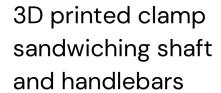

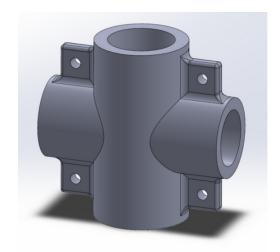








Occupational Therapist testing prototype



Prototype #2

• Testing open handle grip design


 Handle moves relative to the shaft height

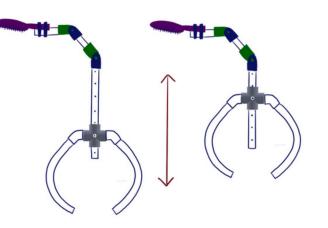

Vertical ends slide & lock up & down shaft




Adjusts in increments & locks with pin

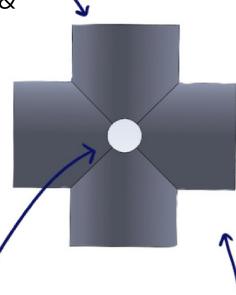

Horizonal openings fit handles securely






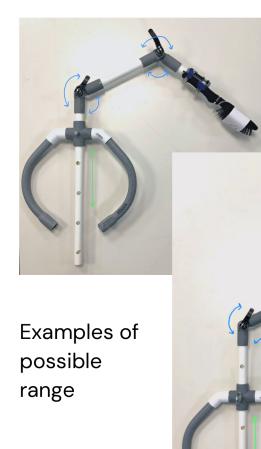

Feedback from Testing


After 1- on-1 feedback and a discussion with a panel of OTs, FOP experts, and the IFOPA, we found these conclusions:

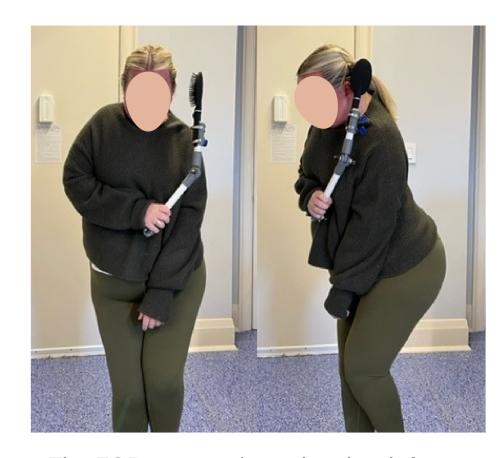

The FOP expert above has her left arm fused by her side, and is able to bend her right forearm up as shown.

The overall height is too large, but she was able to make adjustments to my prototype that worked for her

Next Iteration

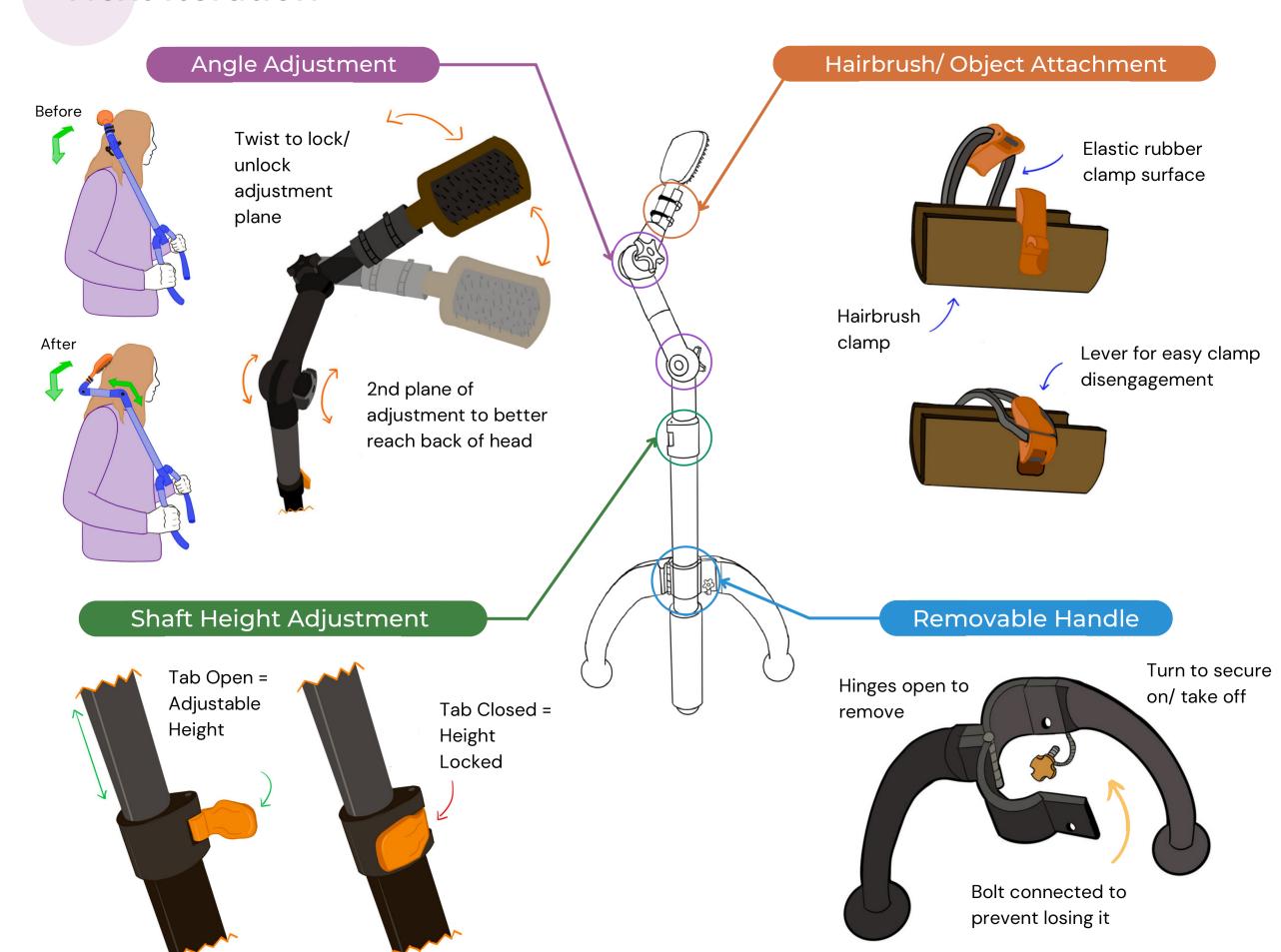

Moving Forward

At the end of the year, Jefferson ID faculty selects devices from The WILL Project to continue developing by assigning them to students in the next cohort. This strategy allows the department to work as a team; building upon prior students' work allows the designs to develop much more detail, so they can ultimately be in the hands of the FOP users!

The Assistive Hairbrush is currently in development by the next student, taking my next steps and beyond as they continue testing, iterating, and progressing the device.

WILL Project

"We Improve Life & Living with FOP"



Project Background

Importance of Industrial Design (ID) & Occupational Therapy (OT) Collaboration

WILL Project

"We Improve Life & Living with FOP"

2019: WILL Project is a collaboration between

Occupational Therapists & Industrial Designers.

2021: IRB Approved data collection during design

process & partnership with Shenkar University, Israel


Jefferson is a worldwide center for FOP treatment,

making this connection possible

ID Impact

- Presence of end-user opinions & usability testing procedures
- Knows how to create beyond single user populations

ID Process

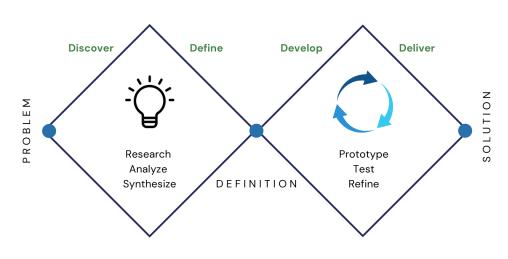

Both Disciplines

- Emphasis on a user centered design process

- Strive to make evidencebased decisions
- Attention to making & iterating

Client- Centered

- Understanding of client's current issues & problems
- Client is the EXPERT
- Only client can identify goals & priorities that are important to them- with help from the therapist







Together, we have a common emphasis on a user centered design process & strive to make evidence-based decisions with research, feedback, & iterations.

OT Impact

- Understand medical & cognitive fundamentals
- Effective solutions for end-users (clients & therapists)

